

Biosteon®

HA/PLLA Composite Anchors

Biologically Inspired Fixation



Introduction

Searching For a Better Option

Bioabsorbable polymers such as PLA and PGA have been used in bodily implants due to their radiolucency for MRI and CT imaging, as well as for their degradation properties. Polymers degrade through simple hydrolysis into materials that can be metabolized by the body¹. However, bioabsorbable polymers are not osteoconductive, have no bone bonding ability and have little potential to be replaced by bone when resorbed^{2,3-5}. Polymers are also susceptible to autocatalytic degradation which can result in acidosis and sterile abscess formation at the site^{6,7}.

Researchers have continued to search for a better option. The focus was to find a material that would mimic the biological function of human bone. Since it was understood that calcium phosphate ceramics are osteoconductive^{8,9}, and calcium phosphates are slightly alkaline, acting to buffer the acidic breakdown products of the polymers, calcium phosphate was added to the polymer to create a composite material.

Biosteon (HA/PLLA)


Biosteon is a composite of hydroxyapatite (HA) and non-crystalline poly-L-lactide (PLLA).

25 % HA

HA is dispersed throughout the Biosteon material providing an osteoconductive material similar to the mineral element of bone^{8,9}. The HA particles have a buffering effect on the acidic (lactic acid) degradation product of the polymer⁴, which helps prevent 'autocatalytic' degradation and premature loss of strength.

75% Amorphous PLLA


The amorphous PLLA in Biosteon provides structural integrity, biocompatibility and a controlled degradation rate¹⁰.

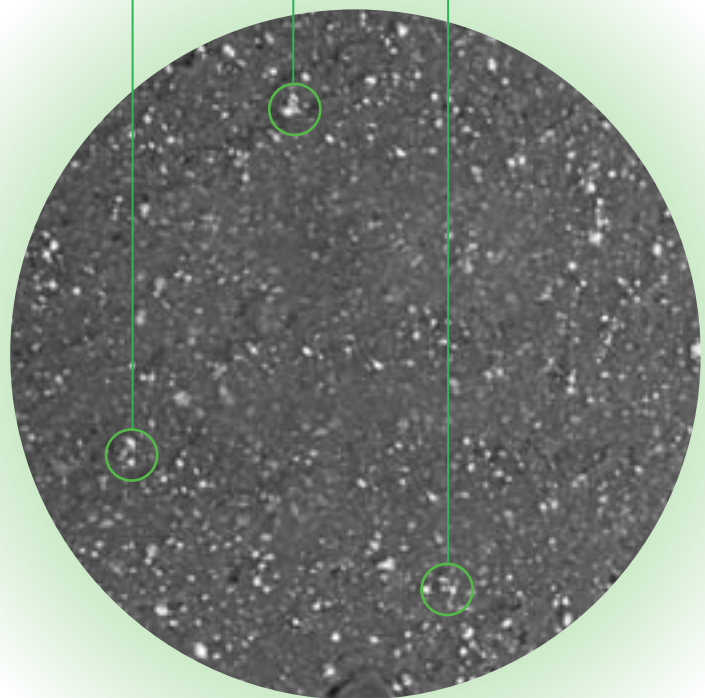

Evidence

Clinical Evidence of Remodeling

Biosteon has been clinically shown to have excellent strength, proven biocompatibility, predictable resorption, osteoconductivity, and remodeling of the screw tract. The graph shows the density in the screw tract has become the same as cancellous bone over the course of the remodeling process¹¹.



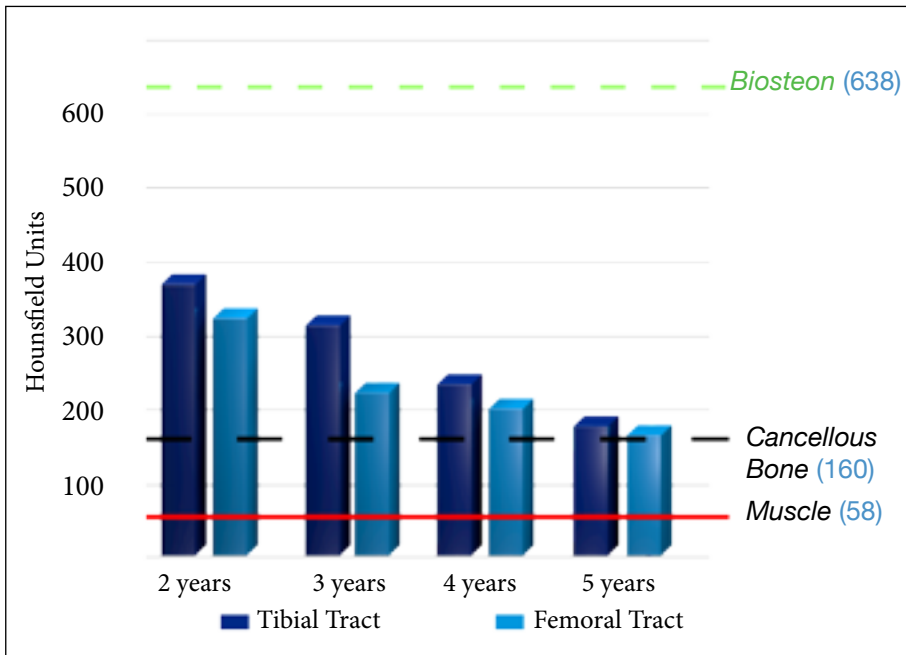


The CT images below show the density of the screw tract at 2 and 5 years¹¹. The Biosteon screw is designed to maintain structural integrity and fixation strength during the healing process and remodel over time.

Biocompatibility

Biosteon HA/PLLA screws have been shown to support bone apposition rather than fibrous tissue formation¹².


The picture below shows an example of hard tissue response using the Biosteon HA/PLLA interference screw. In vivo, 6 months post-implantation, new bone has formed into the contours of the Biosteon screw¹².

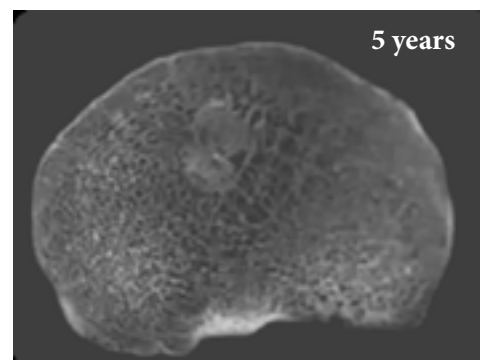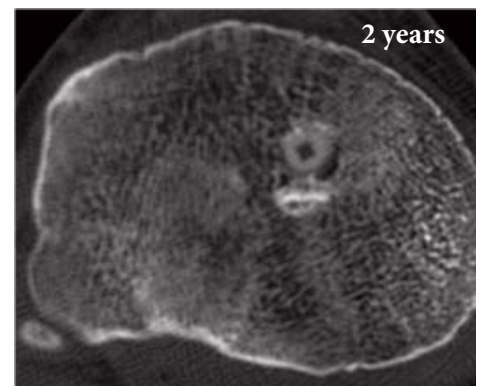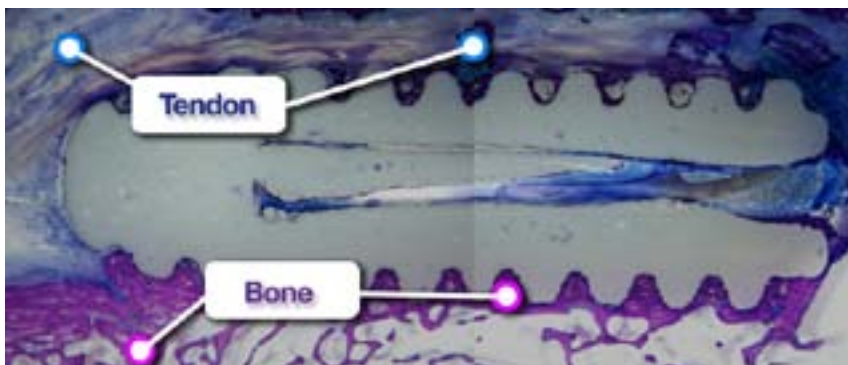


Competitive Advantages

Triple Loop

- Unique design
- Easy suture sliding
- Increased pull-out strength^{13, 14}

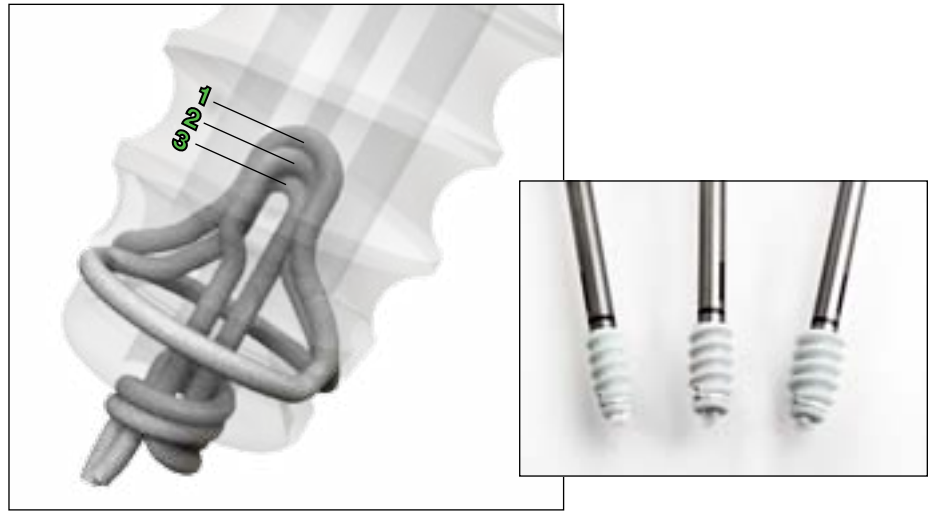

Pull-out Strength^{13, 14}

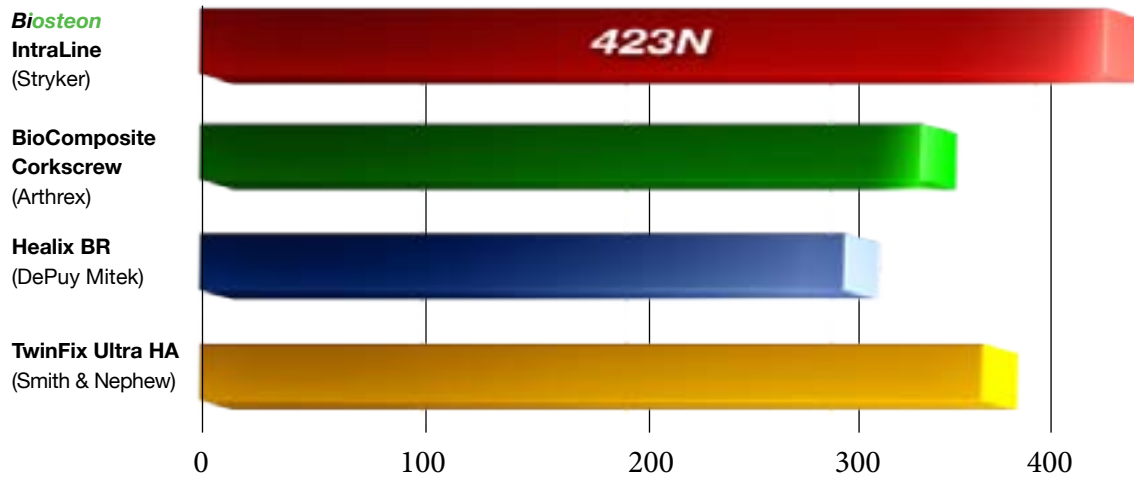

One pound force is approximately equal to 4.5 Newtons. All anchors above are 5.5mm.



		2 - Sutures		3 - Sutures	
		Without Needles	With Needles	Without Needles	With Needles
4.5mm	3910-200-080		x	x	x
5.5mm	3910-200-081		3910-200-082	3910-200-083	x
6.5mm	3910-200-084		3910-200-085	3910-200-086	x

Instrumentation

Ergonomic Handle

4.5mm Punch - Tap



5.5mm Punch - Tap




6.5mm Punch - Tap




Laser Band

The unique laser band allows surgeons to use one tap for both hard and soft bone.

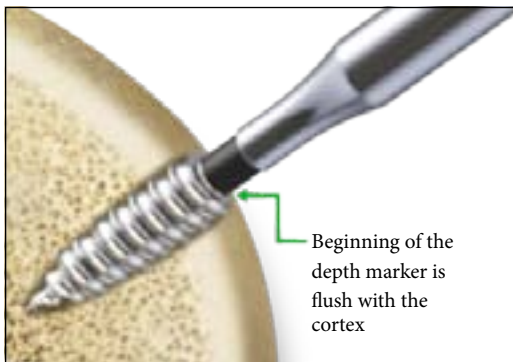

Normal / Soft Bone

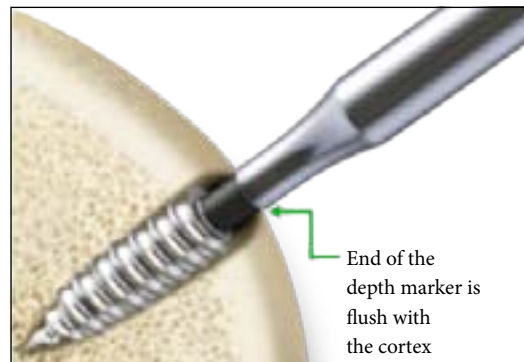

Hard Bone

Composite Anchors

PART NUMBER	DESCRIPTION
3910-200-080	4.5mm Biosteon IntraLine, 2 suture w/out needle
3910-200-081	5.5mm Biosteon IntraLine, 2 suture w/out needle
3910-200-082	5.5mm Biosteon IntraLine, 2 suture w/ needle
3910-200-083	5.5mm Biosteon IntraLine, 3 suture w/out needle
3910-200-084	6.5mm Biosteon IntraLine, 2 suture w/out needle
3910-200-085	6.5mm Biosteon IntraLine, 2 suture w/ needle
3910-200-086	6.5mm Biosteon IntraLine, 3 suture w/out needle
3910-002-045	4.5mm Biosteon IntraLine Punch - Tap
3910-002-055	5.5mm Biosteon IntraLine Punch - Tap
3910-002-065	6.5mm Biosteon IntraLine Punch - Tap

References

- Hunt & Callaghan. "Polymer-hydroxyapatite composite versus polymer interference screws in anterior cruciate ligament reconstruction in a large animal model" *Knee Surg Sports Traumatol Arthrosc* 16:655, 2008.
- Shellock FG, Mink JH, Curtin S et al. (1992) MR imaging and metallic implants for anterior cruciate ligament reconstruction; assessment of ferromagnetism and artifact. *J Magn Reson Imaging* 2:225-228.
- Pihlajamaki H, Kinnunen J, Bostman O (1997) In Vivo monitoring of the degradation process of bioresorbable polymeric implants using magnetic resonance imaging. *Biomaterials* 18:1311-1315.
- Zantop T, Weimann A, Schmidtko R et al (2006) Graft laceration and pullout strength of soft-tissue anterior cruciate ligament reconstruction: an in vitro study comparing titanium, poly-D,L-lactide, and poly-D,L-lactide-tricalcium phosphate screws. *Arthroscopy* 22:1204-10.
- Athanasίου KA, Agrawal CM, Barber FA et al (1998) Orthopaedic applications for PLA-PGA biodegradable polymers. *Arthroscopy* 14:726-37.
- Böstman O. "Intense granulomatous inflammatory lesions associated with absorbable internal fixation devices made of polyglycolide in ankle fracture." *Clin Orthop*, 1992;278:191-199.
- Frokjaer J, Moller BN. "Biodegradable fixation of ankle fractures. Complications in a prospective study of 25 cases." *Acta Orthop Scand*, 1992;63: 434-436.
- Hench. *Bioceramics: From concept to clinic*. *J Am Ceram Soc*. 74:1487, 1991.
- Legeros. Properties of osteoconductive biomaterials: Calcium Phosphates. *CORR* 395:91, 2002.
- Middleton. Synthetic biodegradable polymers as orthopedic devices. *Biomater*. 21:2335, 2000.
- Gasser et al. Absorption and Remodeling of HA-PLLA Composite ACL Interference Screws. *Journal of Arthroscopy*. 27(12); 1671-1678. December 2011.
- Hunt and Callaghan. Polymer-hydroxyapatite composite versus polymer interference screws in anterior cruciate ligament reconstruction in a large animal model. *KSSTA* 16(7); 655-660, (2008).
- Tech Report # RD12-029
- Tech Report # DHFD11168

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

Biosteon is a registered trademark of Biocomposites LTD.

Literature Number: LJPBIB
MS/GS 05/12

Copyright © 2012 Stryker
Printed in USA

325 Corporate Drive
Mahwah, NJ 07430
t: 201 831 5000

www.stryker.com